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WARZIN®

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MD MEDISCH HULPMIDDEL

BEOGD DOELENDE

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MD MEDICAL DEVICE

INTENDED PURPOSE

WARZIN® is used to treat common warts on the feet and soles (plantar warts), warts in the nail plate area, on the hands and other parts of the body (e.g. elbows) in children and adults.

WHEN SHOULD WARZIN® NOT BE USED?

- In the event of hypersensitivity to a component of the product
- For wart-like, mostly congenital skin malformations (moles, liver spots)
- Do not use on warts on the face, warts on or close to mucous membranes (e.g., condyloma, genital warts), hairy warts or warts displaying ulceration.

WARNINGS AND PRECAUTIONS

- As children have extremely delicate skin, special attention must be paid to protecting the surrounding skin sufficiently with petroleum jelly and only applying WARZIN® to specific areas when using the product on children.
- Application to larger areas of skin should be avoided in children and people with severe circulatory disorders (e.g., diabetes) in particular.
- WARZIN® is highly flammable – do not smoke when using it or use it near naked flames.
- Avoid contact with the eyes and mucous membranes.

PREGNANCY AND BREASTFEEDING

No restrictions regarding the use of the product are currently known. In case of doubt, please consult your GP or chemist before using this medical device.

INTERACTIONS

None currently known.

ABILITY TO DRIVE

The use of WARZIN® has no effect on a person's ability to drive.

HOW IS WARZIN® USED?

WARZIN® is intended for use on adults, adolescents and children. Its use does not require any special qualifications and it is mostly applied by patients themselves. In the case of children or patients in care situations, it is recommended that a relative or possibly a medical professional (paediatrician, podiatrist, nurse, etc.) apply the product. Apply to the affected skin area twice per day. Repeat the procedure over 3–8 weeks. Prior to application, the skin should be soaked in warm water for around five minutes and then dried thoroughly. Cover healthy skin around the wart well with petroleum jelly (e.g., Vaseline from Rösch & Handel). Apply plenty of the product to the wart using the brush provided. A milky white, translucent, elastic membrane forms, which should not be removed. Experience has shown that the treated wart heals after 3–8 weeks. Should the described treatment prove unsuccessful, please consult your GP.

WHAT SIDE EFFECTS ARE POSSIBLE?

WARZIN® can irritate healthy skin. In order to avoid this, the skin surrounding the wart should be protected with petroleum jelly – see instructions for use above. If long-lasting symptoms of irritation develop (severe burning sensation or itching, inflammation), discontinue treatment and seek medical advice if necessary. In the event of serious incidents, contact the manufacturer and the responsible healthcare authorities in your country.

HOW SHOULD WARZIN® BE STORED AND DISPOSED OF?

Always seal the bottle tightly again after using the product. Store at room temperature. Keep this medical device out of the reach of children. Do not use this medical device after its use-by date. The use-by date can be found on the packaging.

Do not pour this product down the drain or dispose of it as household waste. These steps

will help to protect the environment.

PACKAGING CONTENTS/ SIZES

WARZIN® 10ml

INGREDIENTS

Colloidium, lactic acid, castor oil

MANUFACTURER

BANO Healthcare GmbH
Annagasse 17, 2500 Baden,
Austria

For further information, please visit our website: www.bano.at

Rösch & Handel is a brand of BANO Healthcare GmbH.

CH REP Runge Med AG,
Saint-Louis-Str. 31, 4056 Basel

VERSION: March 2025

We look forward to your feedback on the product.

Medical device

Manufacturer

Use-by date

Batch code

Consult instructions for use