WARZIN®

GEGEN WARZEN

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Physical effect through peeling

② INSTRUCTIONS FOR USE

Before using this medical device, read the entire patient information leaflet carefully as it contains important information. Always use this medical device exactly as described in this patient information leaflet or as instructed by your doctor or pharmacist.

WHEN IS WARZIN® USED?

WARZIN® is used for the treatment of common warts, warts on the sole of the foot (plantar warts) and on the nail plate (periungual warts).

WHEN MAY WARZIN® NOT BE USED?

- In the event of hypersensitivity to one of the product's components.
- In wart-like, mostly congenital malformations of the skin (birthmarks, moles). Do not apply to warts on the face, on or near mucous membranes (e.g. condylomas, so-called genital warts), hairy warts or warts with ulcers.

WARNINGS AND PRECAUTIONS

Application on larger skin areas should be avoided, especially in small children and persons with severe circulatory disorders (e.g. diabetics).

HOW IS WARZIN® USED?

Apply 2 times daily on the affected skin area; before application, soften the skin area with warm water for about five minutes and then dry well. Cover the healthy skin around the wart with oily skin cream or Vaseline.

Use the provided brush to fully coat the wart; this forms a milky white, translucent, elastic membrane which should not be removed. Experience has shown that the treated wart heals after 3 to 8 weeks.

WHAT ARE THE POSSIBLE SIDE EFFECTS?

WARZIN® may cause skin irritations on healthy skin. To avoid this, the skin surrounding the wart should be covered with an oily skin cream or Vaseline - see instructions for use above. In the event of prolonged irritation (severe burning or itching, inflammation) discontinue treatment and seek medical advice if necessary.

WARNINGS

WARZIN® is highly flammable, therefore do not use near an open fire or while smoking. Always close the bottle tightly. Store at room temperature. Keep this medical device away from children and avoid contact with the eyes and mucosa. Do not use this package after the imprinted expiration date. Do not dispose of this product with household waste.

COMPOSITION

Collodium, lactic acid, castor oil.

MANUFACTURER

BANO Healthcare GmbH
Annagasse 17, 2500 Baden
Austria
www.bano.at

Rösch & Handel is a brand of BANO Healthcare GmbH.

Version of the instructions for use: October 2022

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Rösch & Handel est une marque de BANO Healthcare GmbH.