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 INSTRUCTIONS FOR USE
SILVERIN® STICKS
Silver Nitrate Caustic Sticks
MEDICAL DEVICE

INTENDED PURPOSE
SILVERIN® STICKS are caustic sticks made of wood or plastic with a mixture of silver nitrate and potassium nitrate attached to the end. By adding moisture, the caustic sticks become ready for use. SILVERIN® STICKS enable simple, puntual handling so that the effect is limited to precisely defined areas.
SILVERIN® is used for chemical cauterization in the case of superficial bleeding, nosebleeds, for the treatment of warts, mucous membrane lesions in the oral and genital areas and for excessive wound granulation (proud flesh). Further areas of application include aphthous stomatitis, removal of umbilical cord remains and use in wound management, e.g. in stoma care.
WHEN SHOULD SILVERIN® STICKS NOT BE USED?
SILVERIN® STICKS must not be used if you are allergic to silver nitrate or potassium nitrate.
WARNINGS AND PRECAUTIONS:
• SILVERIN® STICKS should only be used under medical supervision for all unusually discolored or unusually shaped skin changes that cannot be clearly identified as simple warts.
• Do not touch the head of the caustic stick with wet hands.
• Protect your clothing and other objects when handling the caustic stick. Silver nitrate causes brown-black stains that cannot be removed with conventional cleaning agents.
• Clean healthy skin that has been accidentally touched with the head of the caustic stick thoroughly with running water and a sponge. However, a harmless darkening of these areas of skin may still occur. The dark colour fades after about 10–14 days.
• If the caustic stick comes into contact with the eye, rinse immediately with the eyelid open under running water for at least 5 minutes and consult a doctor.
• If used in the mouth, it is essential to ensure that no parts of the head are swallowed. If swallowed, a doctor should be consulted immediately.
• Use in children under 3 years of age should only be carried out by specialised medical personnel.
• For the treatment of nosebleeds in children or an umbilical granuloma, the use of SILVERIN® STICKS 50% is recommended to avoid excessive cauterisation.
• In general, it is recommended to treat only one nostril at a time and only treat the second nostril after around 3 weeks, if necessary.
PREGNANCY AND BREASTFEEDING
If you are pregnant or breastfeeding or if you suspect that you are pregnant or intend to become pregnant, consult your doctor or pharmacist before using this medical device.
INTERACTIONS
There are no known interactions.
EFFECTS ON THE ABILITY TO DRIVE
The use of SILVERIN® STICKS does not affect the ability to drive.
HOW SHOULD SILVERIN® STICKS BE USED?
Always use this medical device exactly as described in this package leaflet. Ask your doctor and pharmacist if you are unsure.
Only treat small areas of skin at a time.
Moisten the head of the caustic sticks with 1–2 drops of clean water. Do not use saline solutions. Make sure that no liquid drips from the stick onto the surrounding area, as this can lead to stains that can't be removed. Gently dab the area to be treated precisely and evenly for 1–2 seconds or roll the stick over the area. Do not press too hard to avoid breaking the head.
The caustic sticks are intended for single use. Repeated use could pose a risk of infection.
Wart removal:
Lay application should only be carried out by adults. Make sure that SILVERIN® is only applied precisely.
The treated area does not usually turn brown-black immediately, but within 24 hours and is then shed from the surrounding tissue over a few days. Observe the changes and repeat the treatment every 2–3 days if necessary. Finish the treatment after a maximum of 14 days. If there is no improvement, consult a doctor. The treatment is painless.
Too frequent applications (without waiting for changes) can lead to irritation or inflammatory reactions. In such cases you should seek medical advice before further applications.
Treatment of bleeding, wounds, excessive wound granulation, aphthous stomatitis and umbilical cord residues:
These applications should only be carried out by doctors and specialised personnel.
The treated area quickly turns grey. A burning sensation

may be felt during treatment. One application is usually sufficient.

WHAT SIDE EFFECTS ARE POSSIBLE?
Hypersensitivity reactions or local irritation of the skin such as itching, slight burning, redness and blisters may occur. If you notice any side effects, consult your doctor or pharmacist. This also applies to side effects that are not listed in the package insert. In the event of serious incidents, contact the manufacturer and the health authority responsible in your country.

HOW SHOULD SILVERIN® STICKS BE STORED AND DISPOSED OF?
Please keep the applicators in the original packaging to protect them from light. Store the package in a dry place.

After opening the package close it well until further use. The heads of the sticks are whitish at first, but may darken in colour over time. This discoloration has no influence on the effect of the product. Keep this medical device out of the reach of children. Do not use this product after the expiry date. The expiry date can be found on the packaging.

Do not dispose of this product in household waste. This helps to protect the environment.

CONTENTS OF THE PACKAGE/PACKAGE SIZES
The package contains caustic sticks made of wood (rigid) or plastic (elastic) with 50% silver nitrate and 50% potassium nitrate (SILVERIN® STICKS 50%) or 75% silver nitrate and 25% potassium nitrate (SILVERIN® STICKS 75%).

Available in different package sizes with different active ingredient concentrations.

10 and 100 pcs./115 mm, 4,5" in length;
10 and 100 pcs./200 mm, 7,9" in length

SILVERIN® STICKS are available in flexible and rigid versions.

COMPOSITION
SILVERIN® STICKS 50%: 50% silver nitrate and 50% potassium nitrate
SILVERIN® STICKS 75%: 75% silver nitrate and 25% potassium nitrate

■ MANUFACTURER
BANO Healthcare GmbH, Annagasse 17, 2500 Baden, Austria, www.bano.at

Further information can be found on our homepage www.bano.at

Rösch & Handel is a brand of BANO Healthcare GmbH.

CH REP
Runge Med AG, Saint-Louis-Str. 31, 4056 Basel

VERSION OF THE INSTRUCTIONS FOR USE:
January 2026

The instructions for use are also available electronically (as a download) at www.bano.at.

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