SILVERIN® STICKS

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INSTRUCTIONS FOR USE
SILVERIN® STICKS
Silver Nitrate Caustic Sticks
MEDICAL DEVICE
EFFECTS ON THE ABILITY TO DRIVE: The use of SILVERIN® Silver Nitrate Sticks does not affect the ability to drive.
3. WHAT IS THE CORRECT WAY TO USE SILVERIN® SILVER NITRATE STICKS?
Always use this medical device exactly as described in the package insert or exactly as instructed by your doctor or pharmacist. Consult your pharmacist if you experience any side effects or require further information or advice. If repeated use of the product does not produce the desired results or produces a negative effect, consult your doctor.
RECOMMENDED DOSAGE: Treat only small areas of skin at a time.
METHOD OF APPLICATION: External use. Prepare a clean and durable surface. Clean the area to be treated. Moisten the tip of the applicator stick by holding the applicator tip briefly under dripping water. The tip will turn white or light grey and is then ready for use. Coat the affected area evenly and by dabbing or rolling the tip over the area several times, taking care to use exact precision. The treated area will turn black within a short time and will then be expelled (shed off) from the surrounding tissue over the period of a few days. Repeat the treatment every 2 – 3 days until the desired results have been achieved. The sticks are intended for single use only.
IN CASE OF OVERDOSE IN THE USE OF SILVERIN® SILVER NITRATE STICKS: Excessive use (without waiting for the treatment to take effect) may cause irritation or inflammatory reactions in the area surrounding the treated area. In such cases, consultation with your health care professional is recommended before any further treatment.
4. WHAT ARE THE POSSIBLE SIDE EFFECTS? Hypersensitivity or local skin irritations such as itching, redness, blisters, and slight burning may occur. If you experience side effects, consult your doctor or pharmacist. This also applies to side effects not listed in the package insert.
5. WHAT IS THE CORRECT WAY TO STORE SILVERIN® SILVER NITRATE STICKS? Silver Nitrate Sticks may not be used if you are allergic to silver nitrate or potassium nitrate. It may take up to 24 hours until the treated area turns black.
WARNINGS AND PRECAUTIONS:
• For all unusually coloured or unusually shaped skin changes that cannot be clearly identified as granulation tissue or normal warts, the SILVERIN® STICKS are only intended for use under medical supervision.
• Do not touch the tip (caustic agent) of the SILVERIN® STICKS when hands are wet or damp.
• Protect your clothing and other objects when handling the applicator stick.
• Silver nitrate causes brown-black stains which cannot be removed with conventional cleaning agents.
• Healthy skin that has been unintentionally touched by the tip of the applicator stick must be thoroughly cleaned using running water and a sponge; despite cleaning, contact with the tip of the stick can result in a harmless dark colouration of the affected areas of the skin. The dark colour fades after about 10 – 14 days.
• If the applicator stick comes into contact with the eye, rinse immediately under running water for at least 5 minutes with the eyelid open and then consult a doctor.
• If used in the area of the mouth, it is imperative to make sure that no parts of the head are swallowed.
CHILDREN: Use for children under 3 years of age should only be carried out following personal medical consultation.
INTERACTIONS: There are no known interactions.
PREGNANCY AND BREASTFEEDING: If you are pregnant or breastfeeding or if you suspect that you are pregnant or intend to become pregnant, consult your doctor or pharmacist before using this medical device.

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Silver Nitrate Caustic Sticks

MEDICAL DEVICE

PLEASE READ THE INSTRUCTIONS FOR USE CAREFULLY BEFORE COMMENCING USE.

Always use this medical device exactly as described in the package insert or exactly as instructed by your doctor or pharmacist. Consult your pharmacist if you experience any side effects or require further information or advice. If repeated use of the product does not produce the desired results or produces a negative effect, consult your doctor.

CONTENTS OF THE INSTRUCTIONS FOR USE:

- What are SILVERIN® STICKS and what are they used for?
- What should be considered before using SILVERIN® Silver Nitrate Sticks?
- What is the correct way to use SILVERIN® Silver Nitrate Sticks?
- What are the possible side effects?
- What is the correct way to store SILVERIN® Silver Nitrate Sticks?
- Package contents and further information

1. WHAT ARE SILVERIN® STICKS AND WHAT ARE THEY USED FOR?

SILVERIN® STICKS are cauterisation sticks made of wood or plastic with a mixture of silver nitrate and potassium nitrate attached to the end. By adding moisture, the silver nitrate sticks become ready for use. SILVERIN® STICKS enable simple, punctual handling so that the effect is limited to precisely defined areas.

SILVERIN® is used for chemical cauterisation of superficial bleeding, nosebleeds, treatment of warts, mucosal lesions in the mouth and genital area and excessive wound granulation (wild flesh). Other areas of application are aphthous stomatitis, removal of umbilical cord remnants and use in wound management, e.g. in stoma care.

2. WHAT SHOULD BE CONSIDERED BEFORE USING SILVERIN® SILVER NITRATE STICKS?

Silver Nitrate Sticks may not be used if you are allergic to silver nitrate or potassium nitrate. It may take up to 24 hours until the treated area turns black.

WARNINGS AND PRECAUTIONS:

- For all unusually coloured or unusually shaped skin changes that cannot be clearly identified as granulation tissue or normal warts, the SILVERIN® STICKS are only intended for use under medical supervision.
- Do not touch the tip (caustic agent) of the SILVERIN® STICKS when hands are wet or damp.
- Protect your clothing and other objects when handling the applicator stick.
- Silver nitrate causes brown-black stains which cannot be removed with conventional cleaning agents.
- Healthy skin that has been unintentionally touched by the tip of the applicator stick must be thoroughly cleaned using running water and a sponge; despite cleaning, contact with the tip of the stick can result in a harmless dark colouration of the affected areas of the skin. The dark colour fades after about 10 – 14 days.
- If the applicator stick comes into contact with the eye, rinse immediately under running water for at least 5 minutes with the eyelid open and then consult a doctor.
- If used in the area of the mouth, it is imperative to make sure that no parts of the head are swallowed.

CHILDREN: Use for children under 3 years of age should only be carried out following personal medical consultation.

INTERACTIONS: There are no known interactions.

PREGNANCY AND BREASTFEEDING: If you are pregnant or breastfeeding or if you suspect that you are pregnant or intend to become pregnant, consult your doctor or pharmacist before using this medical device.

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EFFECTS ON THE ABILITY TO DRIVE: The use of SILVERIN® Silver Nitrate Sticks does not affect the ability to drive.

3. WHAT IS THE CORRECT WAY TO USE SILVERIN® SILVER NITRATE STICKS? Always use this medical device exactly as described in this package insert. Ask your doctor and pharmacist if you need further information.

RECOMMENDED DOSAGE: Treat only small areas of skin at a time.

METHOD OF APPLICATION: External use. Prepare a clean and durable surface. Clean the area to be treated. Moisten the tip of the applicator stick by holding the applicator tip briefly under dripping water. The tip will turn white or light grey and is then ready for use. Coat the affected area evenly and by dabbing or rolling the tip over the area several times, taking care to use exact precision. The treated area will turn black within a short time and will then be expelled (shed off) from the surrounding tissue over the period of a few days. Repeat the treatment every 2 – 3 days until the desired results have been achieved. The sticks are intended for single use only.

IN CASE OF OVERDOSE IN THE USE OF SILVERIN® SILVER NITRATE STICKS: Excessive use (without waiting for the treatment to take effect) may cause irritation or inflammatory reactions in the area surrounding the treated area. In such cases, consultation with your health care professional is recommended before any further treatment.

4. WHAT ARE THE POSSIBLE SIDE EFFECTS? Hypersensitivity or local skin irritations such as itching, redness, blisters, and slight burning may occur. If you experience side effects, consult your doctor or pharmacist. This also applies to side effects not listed in the package insert.

5. WHAT IS THE CORRECT WAY TO STORE SILVERIN® SILVER NITRATE STICKS?

Please keep the applicators in the original packaging to protect them from light. Store the package in a dry place. After opening the packaging, close it well until further use. Keep this medical device out of the reach of children. This medical device may not be used after the expiry date. The expiry date is shown on the back of the packaging. Do not dispose of this product with household waste. Ask your pharmacist how to dispose of the product when it is no longer needed for use. This helps to protect the environment.

6. PACKAGE CONTENTS AND FURTHER INFORMATION:

The packaging containing 10 or 100 applicator sticks made from beech wood (rigid) or plastic (flexible) with a round tip coated with the active substance, 50% silver nitrate and 50% potassium nitrate (SILVERIN® STICKS 50%) or 75% silver nitrate and 25% potassium nitrate (SILVERIN® STICKS 75%).

PACKAGE SIZES:

Available in different package sizes with different drug concentration. Further information can be found on our homepage www.bano.at

10 and 100 pcs./115 mm, 4.5" in length;
10 and 100 pcs./200 mm, 7.9" in length
SILVERIN® STICKS are available in flexible and rigid versions.

MANUFACTURER: BANO Healthcare GmbH, Annagasse 17, 2500 Baden, Austria, www.bano.at

Rösch & Handel is a brand of BANO Healthcare GmbH.

VERSION OF THE INSTRUCTIONS FOR USE: January 2023

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